

MEMORANDUM

To:	Northwell Health Laboratories Clients	
From:	Dr. Dwayne Breining MD, Vice Chair Laboratory Services, Northwell Health Laboratories	
Date:	June 20, 2022	
Re:	Light Sensitive Tests	

Dear Valued Northwell Health Laboratories Clients,

Some analytes degrade when exposed to light. To maintain the integrity of these light-sensitive analytes, proper specimen collection and handling, including protection from light, must be followed.

Therefore, light-sensitive analytes *must* be transported in amber transport tubes. If an amber transport tube is not available, the standard specimen collection tube *must* be covered completely (top to bottom) with aluminum foil and secured with tape.

Test code	Test name	Test Catalog Link
ALA24	Aminolevulinic Acid Urine	https://labs.northwell.edu/test/655391
CAROTENE	Carotene Serum	https://labs.northwell.edu/test/656131
COENZQ10	Coenzyme Q10	https://labs.northwell.edu/test/14714065
LAP	Leukocyte Alkaline Phosphatase	https://labs.northwell.edu/test/738099483
PORB	Porphobilinogen Timed Urine	https://labs.northwell.edu/test/655325
PORP	Porphyrins Fractionated Rand UR	https://labs.northwell.edu/test/655315
PORPH24	Porphyrins 24 Hr Urine	https://labs.northwell.edu/test/655443
PORPPLASMA	Porphyrins, Fractionated Plasma	https://labs.northwell.edu/test/157538296
PORPRU	Porphobilinogen, Qt Rnd UR	https://labs.northwell.edu/test/157538292
РТР	Porphyrins, Total Plasma	https://labs.northwell.edu/test/185882970
VB6	Vitamin B6	https://labs.northwell.edu/test/656805
VITB2	Vitamin B2	https://labs.northwell.edu/test/657015
VITC	Vitamin C	https://labs.northwell.edu/test/657011
VITK	Vitamin K	https://labs.northwell.edu/test/656815
ZINCPROTO	Zinc Protoporphyrin	https://labs.northwell.edu/test/657029

The following list of tests requires protection from light.

Sincerely,

Dr. Dwayne Breining, MD and Your Northwell Health Laboratories Team

If you have any questions, please contact Client Services at (516) 719-1100 or your Northwell Health Sales Representative.